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# ENGLISH

## TECHNICAL SPECIFICATIONS

Medical device Class I Repertoire number: 2109220

Dimensions: 202 x 67x 213 mm

Weight: 850gr

Battery charger: DC 5V/2,0A

Power supply: 110/250 V AC

Battery: 2200mAh

Power: 25W

Charging (0-100%): 2-3 hours

Axi-pure atomizer uses in exclusive:

- Axivirium Bioactive <sup>Med</sup> Class IIa disinfectant for medical devices

- AxiSan sanitizer for surfaces

Bottle capacity: 380ml

Up to 3 bottles of product with a single charging

Surface cover with one bottle: 250mq (tested at min power)

1- On-Off Power switch

2- Power button

3- Type-C charging port

4- Atomization power led

Autonomy:

- min power: 90'

- avg power: 60'

- max power: 30'

Coverage distance:1,5-2,5 mt

Atomization power: 3 level-scale

## INTENDED USE

Axi-pure is a nano sprayer gun machine to disinfect or sanitize surfaces.

By way of example:

Medical practices, hospital, dentists, offices, beauty salons, commercial establishments, Means of transport, Gyms.

Choose Axivirium Bioactive <sup>Med</sup> or AxiSan depending on the intended use.

## HOW TO USE

1.Remove the cap and seal of the disinfectant/sanitizer bottle, insert the pipe and screw **without forcing**.

2.Turn ON "1" button.

3.Press button 2 until to turn on. 6 blue led lights direct the jet towards the less illuminated areas. Turn them off with two quick press button "2".

4.Adjust atomization power pressing button "2"

5.Spray 30-40 cm far from surfaces for 10-12 seconds with sideways movements, leaving it to act.

6. At the initial power level, click 3 times to stop spraying. Surface dries quickly, without any further action.

7. When your disinfection is over, Turn OFF button "1"

## CHARGING

1.Turn OFF button "1".

2.Plug in the power cable to the Type-C port

Atomization process with Axi-pure does not replace cleaning dirty surfaces.

## WARNINGS

Use a soft, moistened cloth and neutral detergent to clean the outer surfaces of the device. The device has a protective film in the buttons and spray area.

## WARNINGS

Do not use the device with un-authorized product. Use the device only on surfaces, spreading uniformly. Do not use on surfaces in contact with foods. Do not wash the machine. Do not nebulize on air, faces, heat sources, open flame, etc. Keep away from children. Incorrect use may cause damage and malfunctions. Do not tip or hold the gun horizontally, it may cause liquid to spill.

## WARRANTY

The appliance is covered by the conventional 1-YEAR warranty from the date of purchase. Components or parts with manufacturing defects will be repaired or replaced free of charge at one of our authorized service centers within the warranty period. The warranty does not cover:

- use the device with un-authorized products;

- repairs or service performed by technicians not authorized by Enproject Medicali srl.

- damage from neglect, detected at fault or negligent;

- defects due to normal wear, not compromising the operation of the device

- damage for collision

- device or components theft

- damage due to climatic event or natural disaster in general.

The warranty will not be honored in the event the make or serial number of the appliance is inexistent, scratched or modified. The following warranty does not include any right to reimbursement for damages, direct or indirect, of any nature towards people, things or animals caused by possible appliance malfunction.

## ERRORS AND MESSAGES

The gun does not deliver product. An air bubble may have formed in the suction channel. Close the hole of the dispensing nozzle (6) with a finger, switch on the device for 2-3 seconds, remove your finger and check the status. Repeat the operation a few times if necessary.

The gun does not turn on. The device will not start if you press and hold the power button (2), even for a short time. It is necessary to press it with a quick click.

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